

1 STEP

Click „View“ the study / studies:

DICOM Library

Anonymize, Share, View DICOM files ONLINE

POWERED BY:


DICOM VIEWER:


- Home Page
 - About us
 - Contact
 - Library usage
 - About Viewer
- TRY NOW**
MedDream VET DICOM Viewer
- TRY NOW**
DICOM Library for Human Healthcare

View DICOM Study:

<https://www.dicomlibrary.vet/?study=1.2.826.0.1.3680043.8.1055.1.20180312165580.83>



View



Copy Link

Download Anonymized DICOM Study:

<https://www.dicomlibrary.vet/?requestType=WADO&studyUID=1.2.826.0.1.3680043.8.1055.1.20180312165580.83>



Download



Copy Link

DICOM Study MANAGEMENT Panel

<https://www.dicomlibrary.vet/?manage=47ddb315cbc2f8b5c8313973abd886>



Delete

We are improving this site, so if you are missing some features or have comments, please [write us a response!](#)
Thank you!
VET DICOMLibrary Team



How to download anonymized study from Viewer?

What can you upload?

You can upload DICOM file or files packed as zip. Zip can contain DICOM files of studies, series and others.

How upload works?

Firstly test if it DICOM file or it will be skipped. Secondly try [anonymize](#) and then upload file to server. Same goes for zip file: try to extract and read file by file, test for DICOM format or skip, [anonymize](#) and upload. All extracting and anonymization is made on your browser.

Share?

You can share uploaded files via social networks: facebook, twitter, send by e-mail and embed DICOM viewer to forum.

Anonymization?


At this point - deletes information about patient and attributes to identify a person in each DICOM file. Anonymization is done in your browser. Do not upload files with information written on image!

Copyright © 2018 by VET DICOM Library. All rights reserved

[Terms of Service](#) | [Privacy Policy](#)

2 STEP

Click „Export“

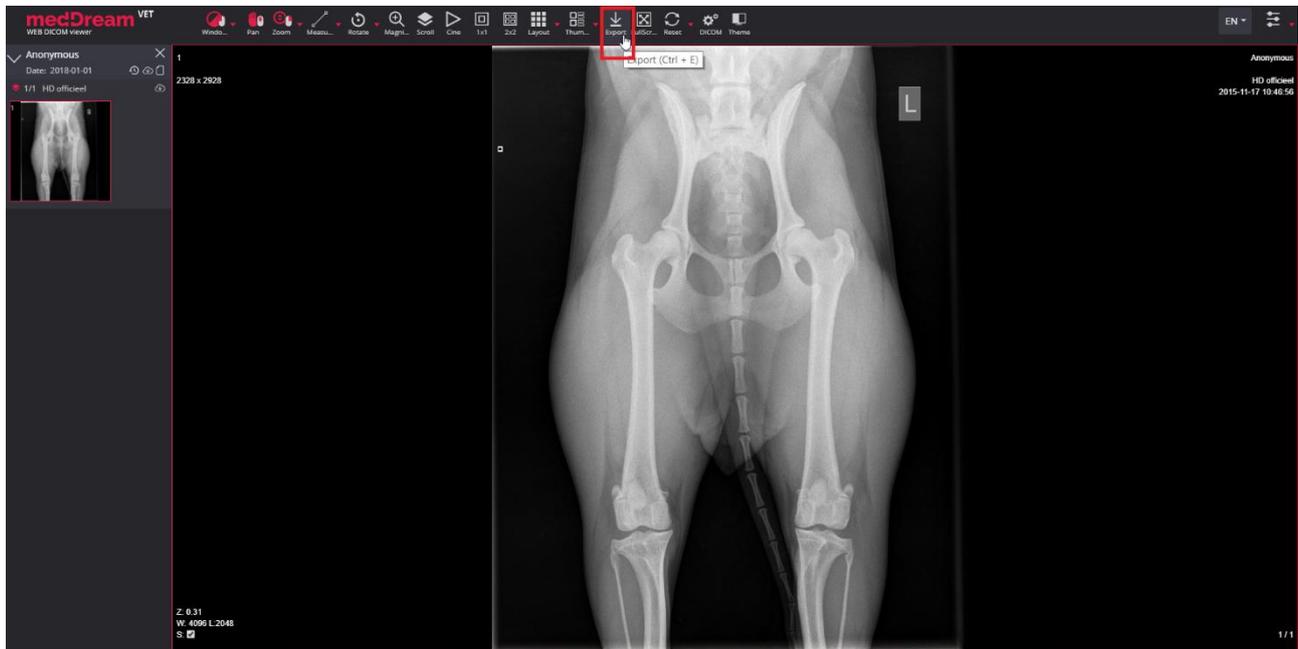

3 STEP

Select „DICOM“ and pick an option „Save Active Image / Video“, then click „Save“.

